
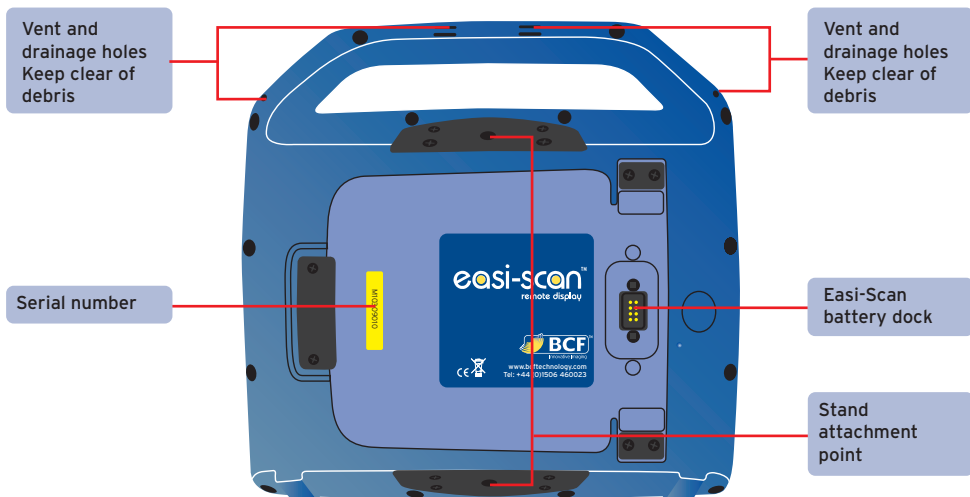


# Quick Guide

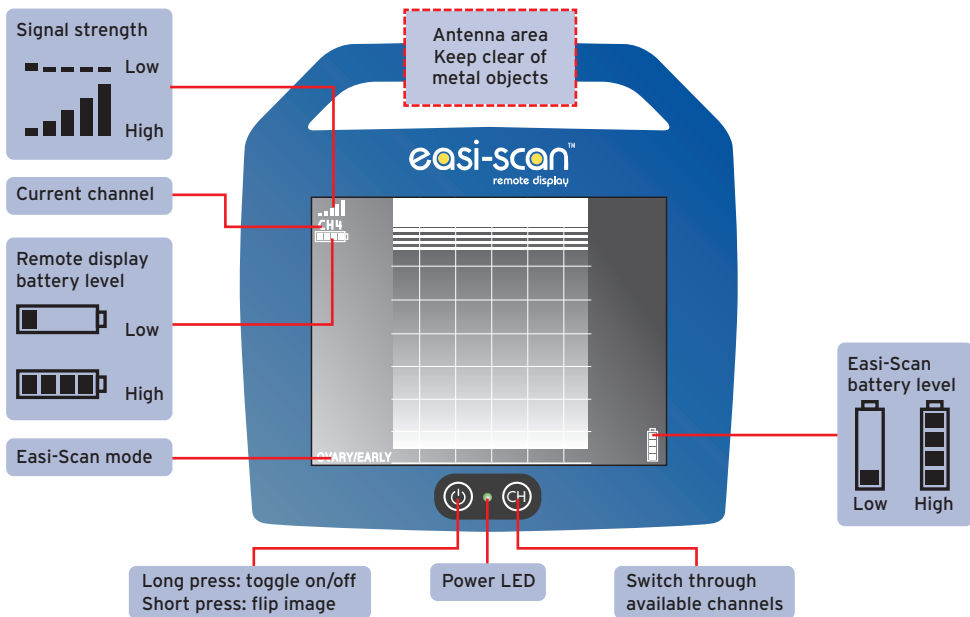

## Care instructions

We have built the Easi-Scan remote display to be rugged and splash resistant, however, it does contain sensitive electronics. With a little care and maintenance you will get many years of excellent imaging.

### Care

The Easi-Scan remote display will be damaged if you:

- Drop it.
- Hose it down.
- Wipe the screen with your hand.

For a long and happy life:

- Clean immediately after use – before any debris dry solid.
- Grit/mud should first be removed by rinsing gently – do not rub screen.
- Dry excess water off the housing with clean soft cloth – do not rub the screen.
- Only wipe the screen with a clean cloth.
- Secure in protective carry case or neoprene sleeve once clean and dry.

## Specifications

Display size:	8.2" or 21cm diagonal
Luminance:	800 cd/m <sup>2</sup>
Contrast ratio:	1000:1
Battery life:	5 hours
Power source:	Easi-Scan battery
Environmental proofing:	IP65
Temperature range:	-20°C to 50°C or -4°F to 122°F
Weight (with Easi-Scan battery):	1.65kg/3.64lb
To be used with:	Easi-Scan and Duo-Scan

Easi-Scan remote display has been designed to operate as a component of BCF ultrasound scanners and as such the combined system conforms with the requirements of European Directive 89/336/EEC relating to electromagnetic compatibility (EMC) and also 2002/95/EC relating to Reduction of Hazardous Substances (RoHS). If you require any further information regarding these directives please contact us directly.

Note: All BCF Technology products are to be used for animal applications only.

UK Tel: +44 (0)1506 460023 USA Tel: +1 (507) 529 8200  
For a list of BCF product distributors visit [www.bcftechnology.com](http://www.bcftechnology.com)

